USER MANUAL



Set up and operational instructions for the Holsim nose snare & ringing model when mounted to the interlocking pig.



Porcine Nose Snare & Ringing Model

Revision A: Porcine Nose Snare & Ringing User Manual.

Product SKU: P500-000

Last Updated: 30 August 2023



'<mark>Parts</mark> List

Equipment Required:

- 1. Snout
- 2. Fasteners for attaching snout.
- 3. Locking dowel

For all other parts or repairs, please contact Holsim through our website www.holsim.co.nz

You will need:


- Interlocking Porcine
- Snare
- Nose Ringing Wire or Rings
- Pliers

All other operational equipment required is to be provided by the user.

PAGE 2 www.holsim.co.nz



The Parts

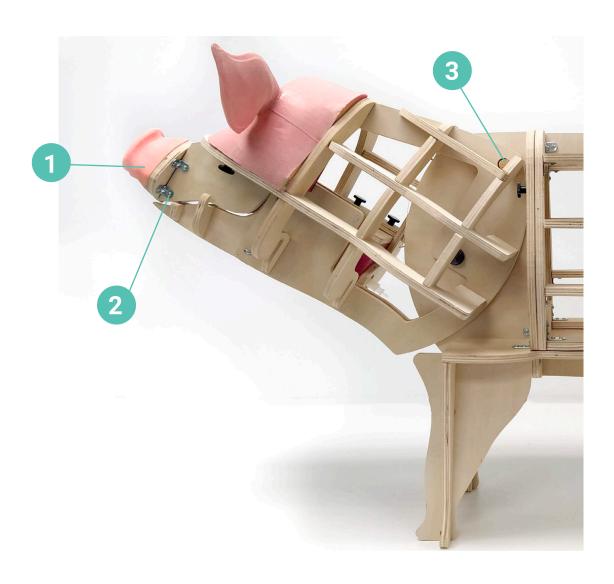

PAGE 3 www.holsim.co.nz



The Set Up



1 Ensure snout is attached

 Ensure snout is securely attached to the interlocking porcine model with screws as shown.



2 Lower Pig Head

 Lower the pig head so that the pig is in the 'eating' position.



3 Ensure jaw is open

• Ensure the jaw is open and spring is stretched open.

PAGE 4 www.holsim.co.nz





4 Check snare

- Locate and check your snare or rope.
- Ensure the loop is at the correct size.



5 Locate snare securely

- Pull the snare rope over the tip of the snout and into the mouth.
- The snare should sit as close to the back teeth as possible.



6 Pull pig head upwards

- Pull the pig's head upwards.
- You will notice that the mouth will clamp shut.

PAGE 5 www.holsim.co.nz



The Set Up



Restrain pig or lock neck

- Restrain the pig by tying off the rope to something nearby.
- Or, lock the pig neck in the 30 degree position with the dowel by aligning the holes in the face plates and neck. Insert the dowel to hold the pig neck in place.



8 Palpate the snout

- Palpate the snout to feel the area you are going to ring.
- This could be at the top edge of the snout, or through the septum.



9 Insert ring/wire

 Push ringing wire through the silicone snout using the tools and techniques you are taught.

PAGE 6 www.holsim.co.nz





10 Loop wire back on itself

• Loop the wire back on itself to form a loop.



11 Tie off wire/close ring

• Tie the ringing wire off tightly and clip excess wire ends.



12 Loosen snare and release

- Loosen the snare and lower the pig's head.
- Release the snare from the pig's mouth

PAGE 7 www.holsim.co.nz



34D Somerset Street, © 07 839 5018

Hamilton, New Zealand PH info@holsim.co.nz