

USER MANUAL



A step by step instruction for setting up,
using and cleaning your Holsim Canine
Intubation Model



Intubation User Model

Revision A:
Intubation Model User
Manual.

Product SKU:
C300-000

Last Updated:
30 August 2023



Parts List

Equipment Required:

- Holsim Canine Intubation Model
- Laryngoscope
- Intubation Tubes (8-10mm)
- Lubricant
- Other Necessary Procedural Equipment
- Holsim Desk Stand (Optional)

You will need:

- Dish Detergent
- Paper Towels
- Air Tight Box



The Dog






The Set Up



1 Place Model on table

- Place your Canine Intubation model on a desk or table.
- Preferably, you can mount the head to our Holsim Desk Stand using the threaded inserts on the back neck plate.

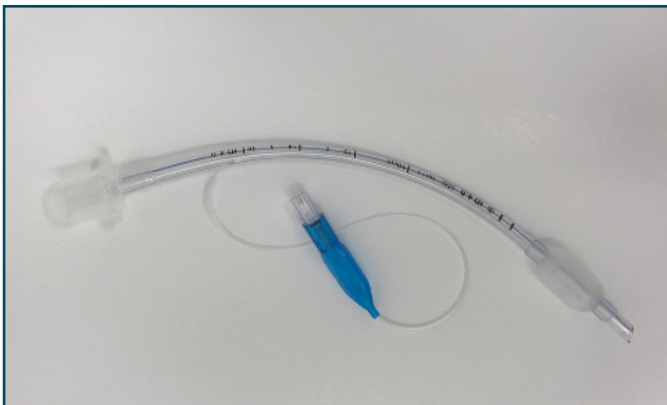

2 Prep intubation tubes

- Prepare your intubation tubes and lubricate them well.
- We recommend using an 8mm diameter tube, but you may want to have a couple of different sizes available.



3 Prep Procedural Equipment

- Prepare a laryngoscope and any other procedural equipment you may need.



The Procedure



1 Pull Tongue out of mouth

- Hold the tip of the tongue with your dominant hand and pull it forward gently out of the mouth.
- Secure with your non-dominant hand and thumb.



2 Insert laryngoscope

- Holding the laryngoscope in your dominant hand, place the tip of it on the back of the tongue, cranial to the base of the epiglottis.
- Once in position, transfer to your non-dominant hand.



3 Insert tube

- Pick up the appropriate tube with your dominant hand and push the tube gently into the dog's mouth towards the back of the tongue.
- Push gently onto the epiglottis to expose the entrance to the trachea and oesophagus.
- Guide the tip of the tube over the dorsal surface of the epiglottis and gently slide it forward into the trachea. It sometimes helps to rotate the tube a little as it enters.




The Cleaning



1 Remove Tube

- Once the procedure has finished, remove the tubes and any other equipment from the dog's mouth as instructed by your tutor.

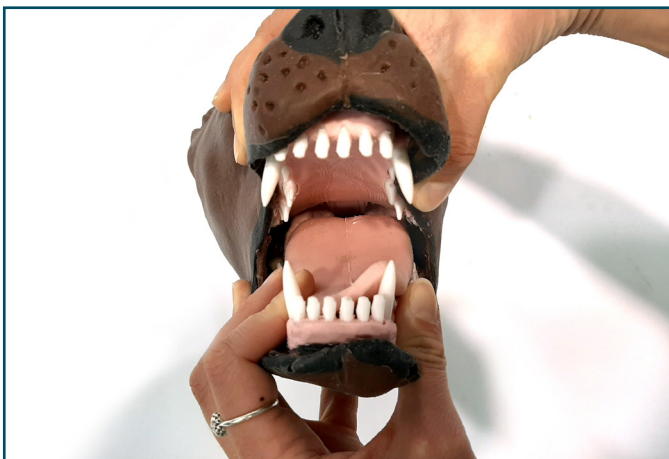

2 Prep intubation tubes

- Wash out the inside of the Canine Intubation Model's mouth using warm water and dish detergent to remove any excess lubricant from the tubes.



3 Dry Model

- Pat the model dry with a paper towel to remove any excess water and moisture.



4 Pack model away

- Once clean and dry, pack away the model away in an air-tight container to keep the model free from dust and dirt.
- Ensure to keep it out of direct sunlight.



34D Somerset Street, Frankton ,
Hamilton, 3204
New Zealand